

*fistula***STOP**[®]

**An innovative
fistula pressure aid**



FistulaStop replaces manual- after dialysis treatment

FistulaStop is comfortable for the patient

- Gentle towards fistulas and grafts
- Applies an even pressure every time
- An easy way to stop the blood flow
- Replaces need for patient to press on fistula following dialysis

FistulaStop provides benefits for the health care workers

- Ergonomic as it replaces pressing by hand
- Easy to clean with alcohol or hot water
- Re-usable
- Can easily be assigned to a particular patient through name-tagged storage box
- One size fits all
- Time saving

FistulaStop is easy to use

- Place a gauze swab between the access of the fistula and the FistulaStop
- Be sure that the bulge covers the fistula access
- Stretch the strap around the arm and secure it
- Remove FistulaStop when the blood flow has stopped and before the patient goes home
- FistulaStop should be changed once a month (if continuously used)

FistulaStop is made in Sweden

www.fistulastop.com



-pressure



Order information

Product	Order code
FistulaStop 1 pair	F 500

CE

FistulaStop is a patient bound product. It is delivered in pairs in a plastic box with a name tag attached on it.

Design protected reg.no. CDR 001614173

*fistula*STOP®



fistula **STOP**[®]

An innovative fistula pressure aid



PRBCG
Medical Supply Specialists

PHONE: +353 (86) 075 6071 E-MAIL: info@prbcg.com
www.prbcg.com www.linkedin.com/company/prbcg